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| | |
|-----|-------------------------------------|
| R1 | 1 × 40 mL – Buffer pH10 / Glutamate |
| R2A | 1 × 10 mL – NAD / Stabilizer |
| R2B | 1 × 0,5 mL – GPT / D-LdH |
| C | 1 × 2 mL - Control solution |

v23M-19032

Reagents preparation:

Reagent N°1: Reagent 1
Reagent is ready to use

Stability of reagents n°1 : see the expire date.

Reagent N°2: Reagent 2A + Reagent 2B
Mix gently 2 vol. of R2A and 0.1 vol. of R2B
Ex: 10ml R2A + 0.5 ml R2B

Stability of reagents n°2 : 30 days.

Automates use:

Range of measurement: The test was developed to determine the concentration in D-lactic acid ranging between 0.01 and 0.25 g/L (0.11-2.5 mmol/L) in the sample with the use of the automate parameters setting supplied by Biosentec.

Procedure of test: For a use on automat, applications are available on request.

Follow the analytical procedure:

- R1 = 200 / Sample = 5
- Incubation 2 min – Lecture 1
- R2 = 40
- Incubation 5 min – Lecture 2
- Multipoint calibration – Linear
- End-point
- Wavelength: I = 340 nm / II = 700 nm

Depending on the used matrix, the calibration could have to be adjusted.

Manual use:

Contact us to get assay protocol.

Storage instructions and reagent stability

The reagents are stable up to the indicated month of expiry, if stored at 2-8 °C, contamination is avoided.
Onboard stability: 5 days

Warnings and precautions

Do not swallow. Avoid contact with the skin and mucous membranes.
See the material security data sheet for more information.

Assay control

D-lactic acid reagents must be validated with the use of the included.

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